QuikRead go software can be updated to a newer version via user interface. Download the newest software version to a USB stick at [www.softwareupdate.quikread.com](http://www.softwareupdate.quikread.com), or order a QuikRead go software USB stick (prod. no. 137540).

**NOTE:** If accu unit is in use, please remove it prior to updating software.

1. Connect the power supply cable and turn the QuikRead go instrument on. Wait for the Start-up to finish and the Main menu to appear.
2. Choose **Settings** → **Measurement flow** → **Maintenance** → **Software update** to start updating.
3. The instrument asks you to insert a USB stick. Insert the QuikRead go software USB stick to one of the USB-ports. There are two USB ports at the back of the instrument and one port on the right side of the instrument.
4. When the instrument has detected the USB stick, it checks if the stick contains valid QuikRead go software. This normally takes less than one minute. The version numbers of the software version on the stick and the one currently in use will be

shown on the display. QuikRead go asks for confirmation before starting update. If you agree, choose **Yes**.

• In case there is no valid software on the USB stick a message *USB stick does not contain a valid software update* appears on the display. Choose **Cancel** to return to the Main page.

5. QuikRead go transfers the new software version from the USB stick. Please wait until software has been completely transferred. This normally takes approx. 6 minutes.

**NOTE: Do not remove the USB stick during the update.**

**Do not shut down QuikRead go or unplug the power supply cable during the update.**

6. After completing the transfer action, QuikRead go shows a message saying it is safe to remove the USB stick. Remove the USB stick and choose **OK**. QuikRead go automatically shuts down and restarts. Finalize the updating procedure by choosing **Restart**. QuikRead go will return to the main menu and is ready for measuring.

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